



**NOBAMED Paul Danz AG**

## **NOBACARE®-classic**

**REF 365303**

### **Product Description and Purpose**

Medical anti-embolism stockings, **knee-length**, individually packed in pairs, without seams due to a special circular knitting method, with inspection opening. The pressure gradient is decreasing from the ankle to the thigh. The white stockings are clearly colour-coded to simplify the identification of the correct size. Disposable measuring tapes and application posters may be obtained free of charge.

### **Contra-Indications**

Medical anti-embolism stockings are not suitable in the case of:

- severe arteriosclerosis or other severe ischaemic vascular diseases including missing foot pulse,
- local skin diseases which would be adversely affected by the stockings, like dermatitis, open ulcera, infections, recent vein ligations, skin graft or gangrene,
- massive peripheral oedema combined with pulmonary oedema,
- severely deformed limbs.

The product NOBACARE® should not be used if there is a known hypersensitivity to one of its ingredients.

### **Composition**

Nylon, elastomer

### **Normative and Legal Requirements**

NOBACARE®-classic medical anti-embolism stockings are class I, rule 1 medical devices according to the MDD, the directive 93/42/EEC and the MDR (EU) 2017/745.

The medical anti-embolism stockings comply with the normative requirements of OENORM ENV 12719.

The product does not contain dangerous toxic substances according to REACH.

It has CE marking and DIN EN ISO 15223-1 and EN 1041 labels on its packaging.

### **Packaging**

Primary packaging: PET-bag  
Secondary packaging: carton made of cellulose

### **Storage**

To be stored in a dry and dust-free environment

### **The product bears following labels and symbols**

