



## NOBAHAFT<sup>®</sup>-fein

REF 470004

### Product Description, Intended use, Application

The elastic fixation bandage NOBAHAFT<sup>®</sup>-fein, white, 4 cm x 4 m, is latex-free and cohesive. It does not adhere to the skin or clothing, but only to itself. The bandage has a fine, smooth structure. It is used under tape bandages, mainly at the ankle joint, but may also be used to fix wound pads.

### Composition

Viscose, polyamide, latex-free polymer

### Contraindications

The product should not be used in the case of a known allergy against the material.

### Note

During application, cohesive bandages should only be stretched slightly as otherwise the arterial blood supply might be disturbed.

### Incident reporting

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### Normative and Regulatory Requirements, Common Standards

Medical Device according to *MDD 93/42/EEC, MDR (EU) 2017/745*.

The product does not contain dangerous toxic substances according to REACH.

### Packaging

Primary packaging: folding box made of cellulose

Secondary packaging: carton made of cellulose

### Symbols used in labelling

Explanation at [www.nobamed.com](http://www.nobamed.com)



Marking on all packaging levels with CE and according to DIN EN ISO 15223-1 and DIN EN ISO 20417

### Storage and Transport

Dry and dustfree

### Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

### Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.