



NOBAHAFT[®]-fein

REF 470006

Product Description, Intended use, Application

The elastic fixation bandage NOBAHAFT[®]-fein, white, 6 cm x 4 m, is latex-free and cohesive. It does not adhere to the skin or clothing, but only to itself. The bandage has a fine, smooth structure. It is used under tape bandages, mainly at the ankle joint, but may also be used to fix wound pads.

Composition

Viscose, polyamide, latex-free polymer

Contraindications

The product should not be used in the case of a known allergy against the material.

Note

During application, cohesive bandages should only be stretched slightly as otherwise the arterial blood supply might be disturbed.

Incident reporting

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Normative and Regulative Requirements, Common Standards

Medical Device according to *MDD 93/42/EEC, MDR (EU) 2017/745*.

The product does not contain dangerous toxic substances according to REACH.

Packaging

Primary packaging: folding box made of cellulose

Secondary packaging: carton made of cellulose

Symbols used in labelling

Explanation at www.nobamed.com



Marking on all packaging levels with CE and according to DIN EN ISO 15223-1 and DIN EN ISO 20417

Storage and Transport

Dry and dustfree

Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.