

## **NOBAMED Paul Danz AG**

## **NOBAHAFT®-fein**

### **REF 470208**

# Product Description, Intended use, Application

The elastic fixation bandage NOBAHAFT®-fein, white, 8 cm x 20 m, is latex-free and cohesive. It does not adhere to the skin or clothing, but only to itself. The bandage has a fine, smooth structure. It is used under tape bandages, mainly at the ankle joint, but may also be used to fix wound pads.

## Composition

Viscose, polyamide, latex-free polymer

## **Contraindications**

The product should not be used in the case of a known allergy against the material.

#### Note

During application, cohesive bandages should only be stretched slightly as otherwise the arterial blood supply might be disturbed.

## Incident reporting

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

# Normative and Regulative Requirements, Common Standards

Medical Device according to MDD 93/42/EEC, MDR (EU) 2017/745.

The product does not contain dangerous toxic substances according to REACH.

## **Packaging**

Primary packaging: folding box

made of cellulose

Secondary packaging: carton made of

cellulose

#### Symbols used in labelling

Explanation at www.nobamed.com









Marking on all packaging levels with CE and according to DIN EN ISO 15223-1- and DIN EN ISO 20417

#### **Storage and Transport**

Dry and dustfree

## Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

### Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.

Product Data Sheet February 16, 2022 [Rev6], replaces: April 01.2020 [Rev5]