



**NOBAMED Paul Danz AG**

## NOBARIP®

**REF 475508**

### Product Description, Intended use, Application

The brown, cohesive bandage (8 cm x 4.5 m) has a stretchability of approx. 60 %. The cohesiveness prevents the slipping out of position of the ready-made bandage. NOBARIP® is easy to tear, which allows a quick individual adaptation of the required length. It is used as a support-, relief- and compression bandage and as a fixation bandage. It is applied on intact skin, open wounds are covered with a wound dressing before.

### Composition

Cotton, elastane, latex free polymer

### Note:

During application, cohesive bandages should not be stretched too much as otherwise the arterial blood supply might be disturbed

### Incident reporting

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### Normative and Regulatory Requirements, Common Standards

Medical Device according to *MDD 93/42/EEC*, *MDR (EU) 2017/745*.

The product does not contain dangerous toxic substances according to REACH.

### Packaging

Primary packaging: folding box made of cellulose

Secondary packaging: carton made of cellulose

### Symbols used in labelling

Explanation at [www.nobamed.com](http://www.nobamed.com)



Marking on all packaging levels with CE and according to DIN EN ISO 15223-1- and DIN EN ISO 20417

### Storage and Transport

Dry and dustfree, protect from sunlight.

### Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

### Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.