



NOBAMED Paul Danz AG

X-RAY RINGTUPFER

REF 871450

Product Description, Intended use, Application

The non-sterile gauze balls, 5.0 cm diameter, with silicone ring, packed per 250 pcs, have an X-ray thread. They are used to absorb blood and body fluids in the operative area or to clean the operation wound from blood and body fluids. The gauze balls have a round shape. Before use the product must be sterilized with a validated procedure (moist heat at 121°C/134°C, 2 to 3 bar, in accordance with DIN EN ISO 17665, radiation in accordance with DIN EN ISO 11137, or ethylene oxide in accordance with DIN EN ISO 11135).

Composition

Cotton, silicone, X-ray-thread contains barium sulfate, PVC, DOTP

Contra-Indications

The product should not be used in the case of a known allergy against the material.

Note

Sterilize before use

Normative and Regulatory Requirements, Common Standards

Medical Device according to *MDD 93/42/EEC*, *MDR (EU) 2017/745*.

The gauze complies with the requirements of DIN EN 14079 type 20.

The product does not contain dangerous toxic substances according to REACH.

Packaging

Primary packaging: PE bag

Secondary packaging: carton made of cellulose

Symbols used in labelling

Explanation at www.nobamed.com



Marking on all packaging levels with CE and according to DIN EN ISO 15223-1- and DIN EN ISO 20417

Storage and Transport

Dry and dustfree

Single use device

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

Disposal

According to locally applicable legal regulations and standards of infection prophylaxis.