



**NOBAMED Paul Danz AG**

# **NOBARIGID<sup>®</sup>-Fingerschiene**

## **Aluminium Finger Splint**

**REF 943225**

### **Product Description, Intended use, Application**

The medical finger splint (2.5 cm x 48 cm) for orthopaedic purposes consists of an aluminium band which is padded on one side with foam material. It is used for the quick and easy immobilization of fingers. The finger splint is easy to form and the length can be adjusted with simple bandage scissors or cast scissors. An adequate bandage is needed to fix the splint.

### **Composition**

Aluminium, polyester-based polyurethan foam

### **Contraindications**

The product should not be used in the case of a known allergy against the material.

### **Incident reporting**

According to MDR (EU) 2017/745, if serious incidents occur in relation to the device, they must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### **Normative and Regulative Requirements, Common Standards**

Medical Device according to *MDD 93/42/EEC, MDR (EU) 2017/745*.

The product does not contain dangerous toxic substances according to REACH.

### **Packaging**

Primary packaging: PE bag

Secondary packaging: carton made of cellulose

### **Symbols used in labelling**

Explanation at [www.nobamed.com](http://www.nobamed.com)



### **Storage and transportation**

To be stored in a dry and dust-free environment

### **Single use device**

Reusing a single use medical device can lead to microbiological danger. Reprocessing for reuse can decrease the product's performance significantly.

### **Disposal**

According to locally applicable legal regulations and standards of infection prophylaxis.