NOBAPROTECT® Face Shield

Gebrauchsinformation NOBAPROTECT® Face Shield

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 Information for use

NOBAPROTECT® Face Shield

READ THESE INSTRUCTIONS CAREFULLY BEFORE USING THIS PRODUCT.

Product: Face shield

Product Code: 672090

Manufacturer: NOBAMED Paul Danz AG, Höltkenstrasse 1-5,
D-58300 Wetter (Ruhr), Germany

Notified Body: Module B

SATRA Technology Europe Ltd, Bracetown Business Park, Clonee, D15YN2P.
Republic of Ireland (NB 2777).

Declaration of Conformity: www.nobamed.com

The products are classified as Category II personal protective equipment (PPE) under the European PPE Regulation (EU) 2016/425. They have been shown to comply with this Regulation and with European Standard EN 166:2001.

The product is designed to provide some protection against contact with airborne droplets which may present a biological hazard. However, always remember that no PPE can provide full protection.

Always use caution when performing a risk-related activity. Never use this face shield to protect against other hazards for which the product is not designed or tested.

PERFORMANCE AND LIMITATIONS OF USE:

The products have been tested in accordance with EN 166: 2001.

FIT AND SIZE:

Wear only properly sized products that are properly adjusted. Products that are too loose may not stay in place and products that are too tight will be uncomfortable.

COMPATIBILITY:

The face shield is suitable for spectacle wearers and can be worn over a visual aid. Similarly, a surgical or FFP mask can be worn under the face shield.

When used with other protective equipment (hearing protection, respiratory protection, etc.), the face shield must be put on in such a way that the protective effects of the other protective equipment are not impaired in their effectiveness or interfere with each other.

To optimise protection, this product should be used with other suitable protective equipment such as gloves, coveralls or gowns.

In this case, consult your supervisor before performing the risk-related activity to ensure that all your protective products are compatible and suitable for your application.

WARNING:

If any part of the face shield that comes into contact with your skin causes an allergic reaction, discontinue use and seek medical advice.

STORAGE AND TRANSPORT:

If the product is wet, allow it to dry completely before storing it. To avoid scratches, make sure that the product is not lying with the viewing window facing downwards. Store it in a well-ventilated place, away from extreme temperatures and direct sunlight. Never place heavy objects on it. Avoid excessive folding and store preferably hanging.

DURABILITY:

The product has a shelf life of 3 years. The lifespan depends on the conditions of use, but the face shield should be discarded and replaced if it is badly scratched or damaged in a way that prevents safe use.

REPAIR:

If the face shield becomes damaged, it will NOT provide optimum protection and the product, should be replaced with a new one. Never use a damaged product.

CLEANING:

Disposable product for personal use. If applicable legislation and infection prevention guidelines permit cleaning, the product can be washed under warm water with ordinary soap. Then dry with a soft cloth. If necessary, the face shield can be cleaned with a non-caustic disinfectant.

LABELLING:

Identification of the manufacturer and the product code/article number.

1 S N - 166 3 S
1 5 2 3 4 5

Meaning of the symbols:

- 1 Optical class
- 2 Abbreviation for resistance to fogging
- 3 Number of EN standard
- 4 Range of use (3= protection against droplets and splashes of liquids)
- 5 Abbreviation for mechanical strength (S= increased robustness)

The book pictogram indicates the reading of these instructions.

The user information is supplied with the product.

DISPOSAL:

In accordance with applicable legislation and in compliance with infection prevention requirements.

Status of information: June 2022 [Rev] 2

REF 672090

LOT XXXXXX

yyyy-mm-dd

yyyy-mm-dd

1 S N-166 3 S
PSA CAT II (EU) 2016/425